


The **Ultimate**  
Surgical Stapling Range

purple surgical

# The *Ultimate* Reloadable Linear Cutter Stapler

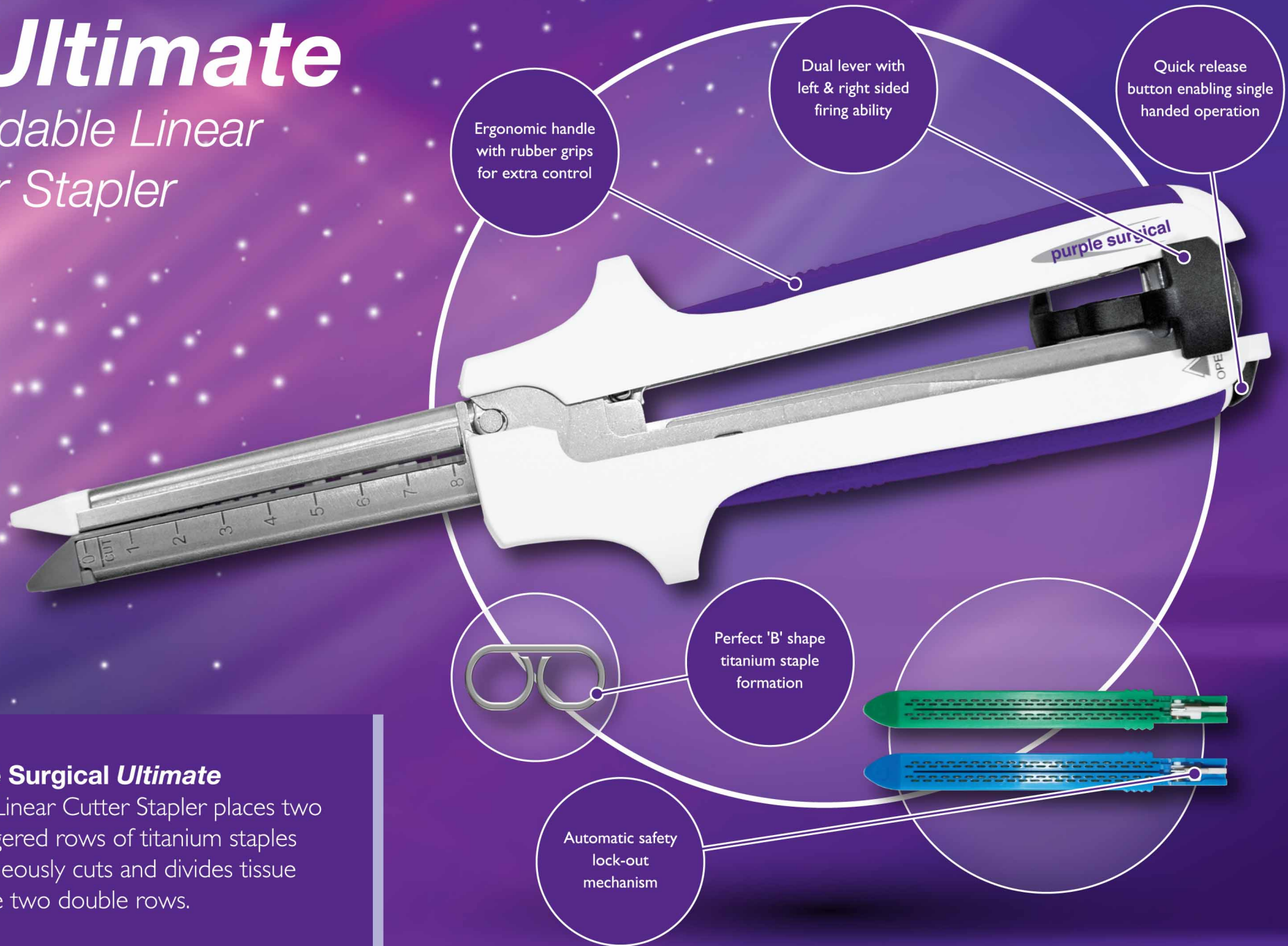

The **Purple Surgical Ultimate** Reloadable Linear Cutter Stapler places two double staggered rows of titanium staples and simultaneously cuts and divides tissue between the two double rows.

The **Purple Surgical *Ultimate*** Reloadable Linear Cutter Stapler is available in 60mm, 80mm and 100mm staple lengths with reloading units available for both standard tissue (3.8mm) and thick tissue (4.8mm). Each device may be reloaded up to 7 times for a maximum of 8 firings per patient. The ergonomic design, with rubber grip inlays and flexible dual lever for left or right handed staple firing, facilitates user comfort and control. Staple formation creates the perfect 'B' shape on each and every firing for improved clinical performance.

Features and benefits include:

- New blade with each cartridge guarantees smooth, precise transection with each firing, reducing the risk of cross infection
- Automatic safety lock-out mechanism incorporated within each cartridge
- Rear hinge allows for one handed operation
- Adjustable firing lever enables you to fire from either side of the device

Anatomy suitable for this device includes:

**General Surgery** - Stomach | Duodenum | Small Bowel | Liver | Colon  
**Gynaecological Surgery** - Fallopian Tubes | Ovaries  
**Urologic Surgery** - Bladder  
**Thoracic Surgery** - Parenchyma

## *Ultimate Reloadable Linear Cutter Stapler*

Product	Code	Colour	Tissue Type	Reload Included	Open Staple Height	Closed Staple Height	Pack Qty
<b>Ultimate</b> 60mm Reloadable Linear Cutter Stapler	PS9LC60B	● Blue	Standard	PS9LCR60B	3.8mm	1.5mm	3 / Box
<b>Ultimate</b> 60mm Reloadable Linear Cutter Stapler	PS9LC60G	● Green	Thick	PS9LCR60G	4.8mm	2.0mm	3 / Box
<b>Ultimate</b> 80mm Reloadable Linear Cutter Stapler	PS9LC80B	● Blue	Standard	PS9LCR80B	3.8mm	1.5mm	3 / Box
<b>Ultimate</b> 80mm Reloadable Linear Cutter Stapler	PS9LC80G	● Green	Thick	PS9LCR80G	4.8mm	2.0mm	3 / Box
<b>Ultimate</b> 100mm Reloadable Linear Cutter Stapler	PS9LC100B	● Blue	Standard	PS9LCR100B	3.8mm	1.5mm	3 / Box
<b>Ultimate</b> 100mm Reloadable Linear Cutter Stapler	PS9LC100G	● Green	Thick	PS9LCR100G	4.8mm	2.0mm	3 / Box

## *Ultimate Linear Cutter Stapler Reloading Units*

Product	Code	Colour	Tissue Type	Cut Length	Rows	Staples	Open Staple Height	Closed Staple Height	Pack Qty
<b>Ultimate</b> 60mm Linear Cutter Stapler Reloading Unit	PS9LCR60B	● Blue	Standard	56mm	4	64	3.8mm	1.5mm	12 / Box
<b>Ultimate</b> 60mm Linear Cutter Stapler Reloading Unit	PS9LCR60G	● Green	Thick	56mm	4	64	4.8mm	2.0mm	12 / Box
<b>Ultimate</b> 80mm Linear Cutter Stapler Reloading Unit	PS9LCR80B	● Blue	Standard	76mm	4	84	3.8mm	1.5mm	12 / Box
<b>Ultimate</b> 80mm Linear Cutter Stapler Reloading Unit	PS9LCR80G	● Green	Thick	76mm	4	84	4.8mm	2.0mm	12 / Box
<b>Ultimate</b> 100mm Linear Cutter Stapler Reloading Unit	PS9LCR100B	● Blue	Standard	96mm	4	104	3.8mm	1.5mm	12 / Box
<b>Ultimate</b> 100mm Linear Cutter Stapler Reloading Unit	PS9LCR100G	● Green	Thick	96mm	4	104	4.8mm	2.0mm	12 / Box

# The *Ultimate*

## Reloadable Linear Stapler with Indicator



Cutting guide offers safety during transection

Low profile design for easy access

Ergonomic, single handed firing lever with rubber grips

Release button cancels firing process at stages 1, 2 or 3

Colour coded indication window:

- 1 Neutral position
- 2 Half closed position
- 3 Tissue compressed between cartridge & anvil
- 4 Staples fired

The **Purple Surgical Ultimate** Reloadable Linear Stapler with Indicator places two staggered rows of titanium staples, and is available in 30mm, 45mm, 60mm and 90mm staple line lengths with reload units available for both standard tissue (3.5mm) and thick tissue (4.8mm).

The **Purple Surgical *Ultimate*** Reloadable Linear Stapler with Indicator may be reloaded up to 7 times for a maximum of 8 firings per patient. The ergonomic design, with rubber grip and single handed firing lever, facilitates user comfort and control. Staple formation creates the perfect 'B' shape on each and every firing for improved clinical performance.

Features and benefits include:

- Cutting guide for enhanced safety during transection
- Colour coded indication window for easy cancelation at stages 1 to 3
- Retaining pin can either be manually or mechanically engaged
- Available in 4 size options (30mm, 45mm, 60mm and 90mm)

Anatomy suitable for this device includes:

**General Surgery** - Oesophagus | Stomach | Duodenum | Small Bowel | Liver | Colon | Pylorus  
**Gynaecological Surgery** - Fallopian Tubes | Ovaries  
**Thoracic Surgery** - Parenchyma | Bronchus

## ***Ultimate*** Reloadable Linear Stapler with Indicator

Product	Code	Colour	Tissue Type	Reload Included	Open Staple Height	Closed Staple Height	Pack Qty
<b><i>Ultimate</i></b> 30mm Reloadable Linear Stapler with Indicator	PS9LSI30B	● Blue	Standard	PS9LSR30B	3.5mm	1.5mm	3 / Box
<b><i>Ultimate</i></b> 30mm Reloadable Linear Stapler with Indicator	PS9LSI30G	● Green	Thick	PS9LSR30G	4.8mm	2.0mm	3 / Box
<b><i>Ultimate</i></b> 45mm Reloadable Linear Stapler with Indicator	PS9LSI45B	● Blue	Standard	PS9LSR45B	3.5mm	1.5mm	3 / Box
<b><i>Ultimate</i></b> 45mm Reloadable Linear Stapler with Indicator	PS9LSI45G	● Green	Thick	PS9LSR45G	4.8mm	2.0mm	3 / Box
<b><i>Ultimate</i></b> 60mm Reloadable Linear Stapler with Indicator	PS9LSI60B	● Blue	Standard	PS9LSR60B	3.5mm	1.5mm	3 / Box
<b><i>Ultimate</i></b> 60mm Reloadable Linear Stapler with Indicator	PS9LSI60G	● Green	Thick	PS9LSR60G	4.8mm	2.0mm	3 / Box
<b><i>Ultimate</i></b> 90mm Reloadable Linear Stapler with Indicator	PS9LSI90B	● Blue	Standard	PS9LSR90B	3.5mm	1.5mm	3 / Box
<b><i>Ultimate</i></b> 90mm Reloadable Linear Stapler with Indicator	PS9LSI90G	● Green	Thick	PS9LSR90G	4.8mm	2.0mm	3 / Box

## ***Ultimate*** Linear Stapler Reloading Units

Product	Code	Colour	Tissue Type	Rows	Staples	Open Staple Height	Closed Staple Height	Pack Qty
<b><i>Ultimate</i></b> 30mm Linear Stapler Reloading Unit	PS9LSR30B	● Blue	Standard	2	11	3.5mm	1.5mm	12 / Box
<b><i>Ultimate</i></b> 30mm Linear Stapler Reloading Unit	PS9LSR30G	● Green	Thick	2	11	4.8mm	2.0mm	12 / Box
<b><i>Ultimate</i></b> 45mm Linear Stapler Reloading Unit	PS9LSR45B	● Blue	Standard	2	15	3.5mm	1.5mm	12 / Box
<b><i>Ultimate</i></b> 45mm Linear Stapler Reloading Unit	PS9LSR45G	● Green	Thick	2	15	4.8mm	2.0mm	12 / Box
<b><i>Ultimate</i></b> 60mm Linear Stapler Reloading Unit	PS9LSR60B	● Blue	Standard	2	21	3.5mm	1.5mm	12 / Box
<b><i>Ultimate</i></b> 60mm Linear Stapler Reloading Unit	PS9LSR60G	● Green	Thick	2	21	4.8mm	2.0mm	12 / Box
<b><i>Ultimate</i></b> 90mm Linear Stapler Reloading Unit	PS9LSR90B	● Blue	Standard	2	33	3.5mm	1.5mm	12 / Box
<b><i>Ultimate</i></b> 90mm Linear Stapler Reloading Unit	PS9LSR90G	● Green	Thick	2	33	4.8mm	2.0mm	12 / Box

# The *Ultimate* Circular Stapler with Dual Safety



The **Purple Surgical *Ultimate*** Circular Stapler with Dual Safety places a circular staggered double row of titanium staples in the tissue and resects excess tissue to create a circular anastomosis.

The **Purple Surgical Ultimate** Circular Stapler with Dual Safety is available in 5 size options, indicated by colour coded anvil heads. 3 staple sizes are available to provide optimal tissue thickness matching. The ergonomic design and textured firing handle facilitates user comfort and control. Staple formation creates the perfect 'B' shape staple when the red line is positioned centrally within the green firing indicator window.

Features and benefits include:

- Dual safety lock-out mechanism prevents misfiring
- Atraumatic Dilating Tip Trocar for reduced trauma
- Fin design Wing Nut for easy counting of full or half turns
- Available in 5 size options with 3 open staple heights and variable closed staple heights for greater choice and flexibility

Anatomy suitable for this device includes:

**General Surgery** - Oesophagus | Stomach | Duodenum | Colon | Rectum



## Ultimate Circular Stapler with Dual Safety

Product	Code	Colour	Lumen	Shaft Length	Rows	Staples	Open Staple Height	Closed Staple Height	Pack Qty
<b>Ultimate</b> 24mm Circular Stapler with Dual Safety	PS9CS24S	Yellow	15mm	18.5cm	2	18	4.5mm	1.8mm	3 / Box
<b>Ultimate</b> 26mm Circular Stapler with Dual Safety	PS9CS26S	White	17mm	18.5cm	2	20	4.8mm	2.0mm	3 / Box
<b>Ultimate</b> 29mm Circular Stapler with Dual Safety	PS9CS29S	Blue	20mm	18.5cm	2	24	4.8mm	2.0mm	3 / Box
<b>Ultimate</b> 32mm Circular Stapler with Dual Safety	PS9CS32S	Green	24mm	18.5cm	2	30	5.0mm	2.2mm	3 / Box
<b>Ultimate</b> 34mm Circular Stapler with Dual Safety	PS9CS34S	Purple	24.5mm	18.5cm	2	32	5.0mm	2.2mm	3 / Box

# The *Ultimate* Circular Stapler with Dual Safety & Tilt Top Anvil



The **Purple Surgical *Ultimate*** Circular Stapler with Dual Safety & Tilt Top Anvil places a circular staggered, double row of titanium staples in the tissue and resects excess tissue to create a circular anastomosis.



The **Purple Surgical *Ultimate*** Circular Stapler with Dual Safety & Tilt Top anvil is available in 4 size options indicated by colour coded anvil washers. 3 staple sizes are available to provide optimal tissue thickness matching. The ergonomic design and textured firing handle facilitates user comfort and control. Staple formation creates the perfect 'B' shape staple when the red line is positioned centrally within the green firing indicator window.

Features and benefits include:

- Dual safety lock-out mechanism prevents misfiring
- Vented, low profile tilt top anvil releases vacuum and minimises the potential of tissue trauma on removal
- Angled wing nut for easy counting of full and half turns
- Available in 4 size options with 3 open staple heights and variable closed staple heights for greater choice and flexibility

Anatomy suitable for this device include:


**General Surgery** - Oesophagus | Stomach | Duodenum | Colon | Rectum



## ***Ultimate*** Circular Stapler with Dual Safety & Tilt Top anvil

Product	Code	Colour	Lumen	Shaft Length	Rows	Staples	Open Staple Height	Closed Staple Height	Pack Qty
<b><i>Ultimate</i></b> 24mm Circular Stapler with Dual Safety & Tilt Top Anvil	PS9CSA24S	● Yellow	15mm	18.5cm	2	18	4.5mm	1.8mm	3 / Box
<b><i>Ultimate</i></b> 26mm Circular Stapler with Dual Safety & Tilt Top Anvil	PS9CSA26S	○ White	17mm	18.5cm	2	20	4.8mm	2.0mm	3 / Box
<b><i>Ultimate</i></b> 29mm Circular Stapler with Dual Safety & Tilt Top Anvil	PS9CSA29S	● Blue	20mm	18.5cm	2	24	4.8mm	2.0mm	3 / Box
<b><i>Ultimate</i></b> 32mm Circular Stapler with Dual Safety & Tilt Top Anvil	PS9CSA32S	● Green	24mm	18.5cm	2	30	5.0mm	2.2mm	3 / Box

# The *Ultimate* Haemorrhoidal Circular Stapler with Dual Safety & Transparent Cartridge

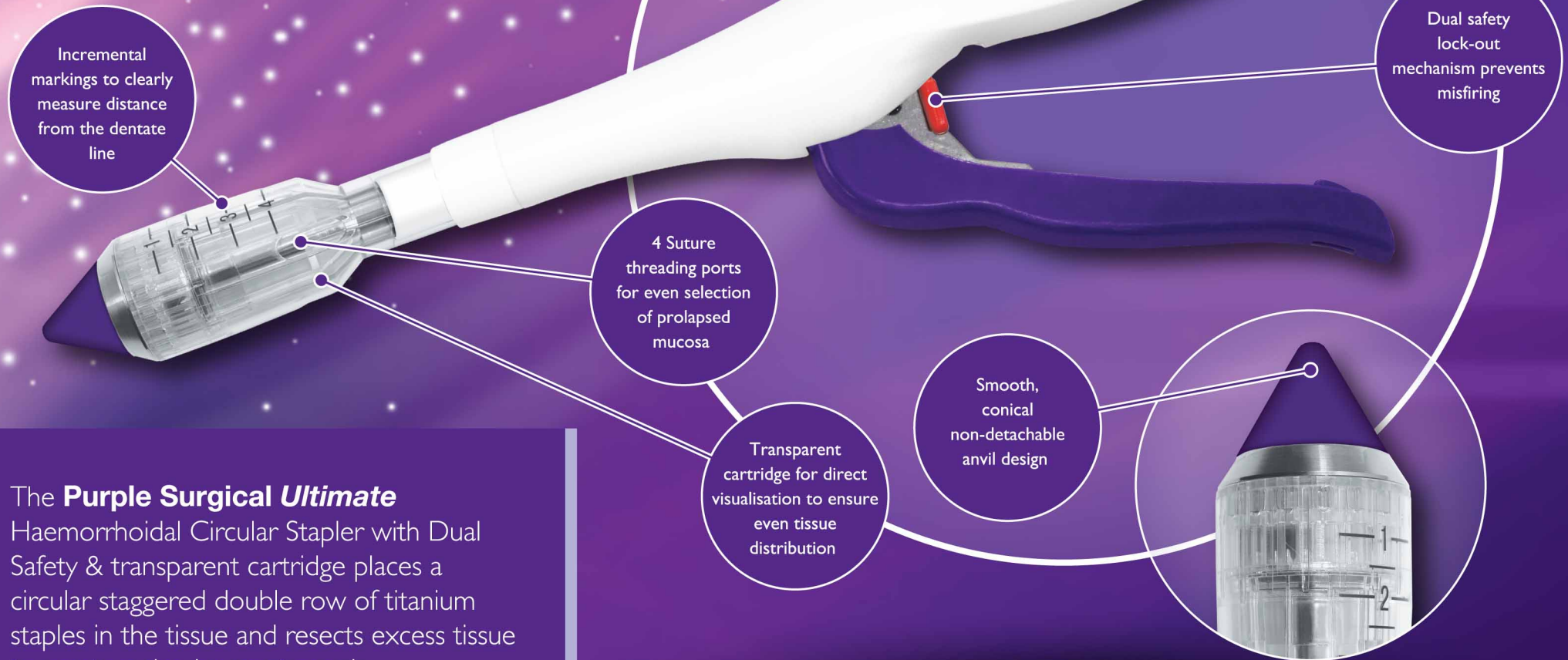

The **Purple Surgical Ultimate** Haemorrhoidal Circular Stapler with Dual Safety & transparent cartridge places a circular staggered double row of titanium staples in the tissue and resects excess tissue to create a circular anastomosis.

The **Purple Surgical *Ultimate*** Haemorrhoidal Circular Stapler with Dual Safety & transparent cartridge is designed for use in the surgical treatment of haemorrhoidal disease. Available in two diameters; 32/33mm and 33/34mm and supplied with 5 accessories including both a clear circular anal dilator and clear butterfly anal dilator.

Features and benefits include:

- Dual safety lock out mechanism prevents misfiring
- Fixed anvil to eliminate risk of detachment from the cartridge
- Transparent cartridge for direct visualisation of tissue within the cartridge
- Double staggered row of staples, for secure staple line formation

The ***Ultimate*** Haemorrhoidal Circular Stapler is supplied with the following 5 accessories:



*Clear Circular Anal Dilator*



*Clear Butterfly Anal Dilator*



*Purse-String Suture Anoscope*



*Anal Dilator Obturator*



*Suture Threader*

## ***Ultimate*** Haemorrhoidal Circular Stapler with Dual Safety & Transparent Cartridge

Product	Code	Rows	Staples	Knife Diameter	Open Staple Height	Closed Staple Height	Pack Qty
<b><i>Ultimate</i></b> 32/33mm Haemorrhoidal Circular Stapler	PS9HCS32/33	2	32	22mm	3.8mm	1.5mm	3 / Box
<b><i>Ultimate</i></b> 33/34mm Haemorrhoidal Circular Stapler	PS9HCS33/34	2	34	24mm	3.8mm	1.5mm	3 / Box



**Purple Surgical International Limited | Purple Surgical UK Limited**  
2 Chestnut House, Farm Close, Shenley, Herts WD7 9AD, England  
Tel: +44 (0)1923 839333 Fax: +44 (0)1923 839444 Email: [mail@purplesurgical.com](mailto:mail@purplesurgical.com)

[www.purplesurgical.com](http://www.purplesurgical.com)